
lisa
sign●lux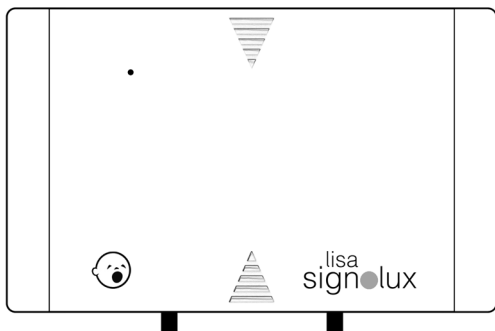

Baby Transmitter

DE

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Seite 2

EN

User manual

Page 14

FR

Mode d'emploi

Page 24

NL

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Pagina 34

FI

Käyttöohje

Sivu 44

DA

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Side 54

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Figure 1: View of top / side / front


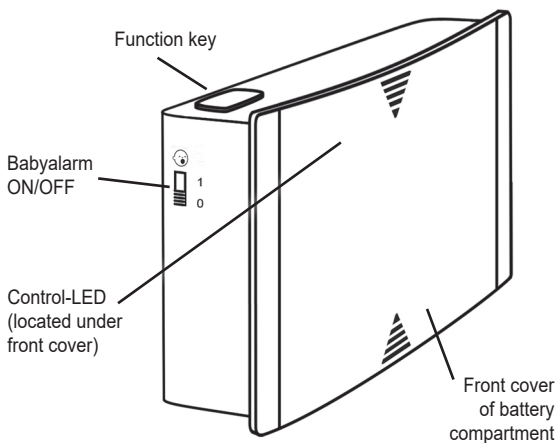
Figure 2: View of underside

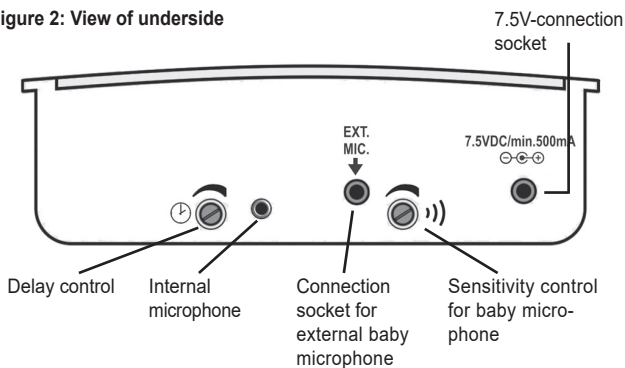The *Lisa-Signolux* »Baby Transmitter« helps people with hearing problems to compensate for their disability. It enables to perceive the cries of a baby or toddler in the home through visible signals.

The »Baby Transmitter« is part of a medical system and is compatible with all devices of the *Lisa-Signolux* wireless system.

Package contents

Please check if the following parts are included:

- | | |
|---------------------|------------------------|
| - Baby Transmitter | - 3 x battery type AAA |
| - Power supply unit | - Wall mount |
| - User manual | - Warranty card |

If the packaging or the packaging seal (round transparent sticker) is damaged, check the condition of the parts. If parts are missing or damaged, please contact your specialist dealer or the manufacturer directly.

Immediate safety information

- Please read these instructions carefully and completely before setting up and using the appliance. Be sure to observe the other warnings and safety instructions on page 22 and page 23.
- Keep this user manual in a safe place so that it is accessible to other users at all times. Every main user must have access to it.
- Keep *Lisa-Signolux* devices out of the reach of children! This packaging contains small parts - there is a risk of choking and suffocation!
- The flashes of light emitted by the receiver devices can trigger epileptic seizures in people with photosensitive epilepsy (PSE).
- After setting up the devices, test the function of the alerting system!

Function / Intended use

The Baby Transmitter detects a child's crying by means of a microphone and in consequence generates radio signals.

The radio signals are transmitted to all *Lisa-Signolux*-receivers within range.


The receivers react to the radio signals by emitting light-, vibration- or acoustic alerts (depending on the type of receiver).

Setup


Inserting / replacing the power failure backup batteries

To insert the 3 AAA-batteries, first remove the front cover for access to the battery compartment:


- Push the front plastic cover upwards. A certain resistance, caused by the latching, has to be overcome. (Fig. 3)
- The plastic cover can now be taken off, the battery compartment is accessible.
- Insert the 3 AAA-batteries. Ensure the correct alignment of the batteries. (Fig. 4)
- Slide the cover down to the snap-in point. The cover is now fixed again. (Fig. 5)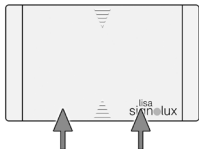

(Fig. 3)

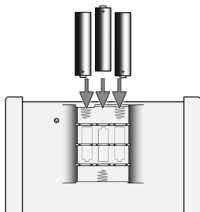(Fig. 4)

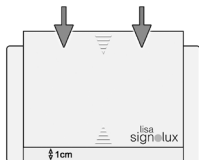(Fig. 5)

Mains operation

Plug the connector at the end of the 7.5 V power supply unit into the socket provided on the underside of the baby transmitter. Then plug the power supply unit into a standard 230 V domestic socket.

Power failure backup

Please note that the device is designed for mains operation. The batteries are intended solely as an emergency power supply in the event of a power failure. The batteries will last for a few days.

Automatic pairing

Lisa-Signolux devices can be used together immediately and without prior pairing when delivered. By default, the Baby Transmitter will be indicated by receiver devices as follows:

- when pressing the function button as „Doorbell 2“ 
- when the internal microphone is triggered with „baby alarm“ 
- when an external baby microphone is triggered with „baby alarm“ 

Important notice 1:

If the transmitter is to be displayed with an event icon of your own choice, this can be done as described in the chapter „*Pairing and assigning to event icons*“. **However, this causes the according receiver to lose the automatic pairing function and every transmitter previously indicated on the receiver must then be individually paired.**

Important notice 2:

Should there be repeated false alerts when using automatic pairing, this may hint at a neighbouring *Lisa-Signolux* installation causing interference. In this case, the problem can be solved by individually pairing all transmitter and receiver units within the setup (see „*Pairing and assigning to event icons*“).

Pairing and assigning to event icons

The Baby Transmitter can be indicated with an event icon of your own choice.

To do this, press and hold the function key of the respective receiver for a few seconds. The event icons of the receiver light up one after the other. Release the function key as soon as the desired icon is lit up.

Now press the function key of the Baby Transmitter.

Wait for approx. 20 seconds and then test the setup by pressing the transmitter's function key. The receiver should now react, showing the according icon. Up to 8 transmitters can be registered per icon.

Setup as a baby alarm

The Baby Transmitter can register a crying child and then transmit an according signal to all *Lisa-Signolux*-receivers within reach.

This is done via the built-in microphone or the external baby microphone cable (item number A-2691-0, available as an accessory).

Place the transmitter near the head of the child's bed. A wall bracket is included in the packaging for wall mounting.

Alternatively, the A-9213-0 and A-9217-0 mounting brackets can also be used to set up the transmitter.

For reasons of health and safety: Make absolutely sure that the child cannot reach the transmitter or, if applicable, the external microphone and the corresponding cables!

Using an external baby microphone

If, for reasons of space or other reasons, the transmitter cannot be placed near the child's bed, it is possible to connect the baby microphone cable (item number A-2691-0, available as an accessory).

Plug the baby microphone cable into the connection socket on the underside of the transmitter. This will deactivate the internal microphone. Attach the baby microphone cable near the head of the child's bed. A self-adhesive Velcro strip is included with the microphone cable for this purpose.

The plug-in microphone A-2976-0 can also be used, but its function is already covered by the factory-installed internal microphone.

Sensitivity setting

To ensure reliable detection by the transmitter, the sensitivity control should be set so that it responds reliably to the baby's crying.

The sensitivity control is used to adjust the internal microphone and any connected external baby microphone.

Procedure:

- Turn the sensitivity control knob to the most sensitive position (turn the sensitivity control knob clockwise).

- If the transmitter is triggered by irrelevant noises (such as sighing or footsteps, etc.), gradually reduce the sensitivity of the transmitter (turn the sensitivity control knob anticlockwise) until the transmitter no longer reacts to the unwanted noise.

However, the exact setting should be adjusted to your personal requirements by changing the setting value several times.

The baby alarm function can be conveniently switched on and off using the baby alarm switch (Fig. 1).

Setting the release delay

A trigger delay can be set to further avoid unnecessary alerts.

Function explanation:

An average noise level is recorded within an adjustable time between 0 and 15 seconds. This means that an alert is only triggered by a sustained sound such as a child crying, and not by shorter sounds such as giggling or sighing.

Delay knob turned anticlockwise to the lowest setting:

Delay = 0 seconds – the transmitter triggers immediately when the volume set via the sensitivity control is reached.

Delay control knob turned clockwise to maximum setting:

Delay = 15 seconds – the transmitter only triggers 15 seconds after reaching the average volume set via the sensitivity control.

Here too, the exact setting should be adjusted to your personal requirements by making multiple changes and testing the setting value.

Warnings and safety instructions

This chapter lists further important information for safe handling and other conditions for operation.

Safety instructions for batteries and power supply

- Only use the specified battery type AAA (LR03)! There is a risk of explosion if the wrong type of battery is used.
- Batteries must not be exposed to high or low extreme temperatures and low air pressure during use, storage or transport at high altitude.
- Disposing of batteries in fires, ovens or by mechanical crushing and cutting may cause fires or explosions. Exposing batteries to extremely high temperatures may cause an explosion or leakage of flammable liquid or gas.
- Batteries exposed to extremely low air pressure may explode or emit flammable liquid or gas.
- Use only the original power supply with the transmitter.
- Remove the batteries from the unit if it will be out of use for a longer period of time (> 3 months).
- If the battery condition is insufficient, event messages from the system may be missed!
- Lay the cable of the power supply unit in such a way as to avoid tripping hazards and possible damage to the cable.
- Rechargeable batteries and regular batteries are toxic and must not be swallowed! Keep them out of the reach of children and pets. In case of emergency, seek appropriate medical attention immediately.

Information on product safety

- Please observe these instructions to avoid damage to the unit and possible invalidation of the warranty.
- The device is designed for use in living areas. The unit must not be exposed to direct sunlight for long periods of time and must also be protected from excessive heat, moisture or strong mechanical shocks.
- The device is not protected against splash water. Do not place any liquid-filled objects next to the device.
- Only place the unit on flat surfaces. Do not drop the unit. Dropping the unit on a hard surface can damage or destroy it.
- Do not disassemble the device or the power supply unit; there is a risk of electric shock. Opening and modifying the unit will immediately void the warranty. Only use accessories from Humantechnik.
- Do not place the receivers of the system in extremely bright locations, such as sunlit windowsills, as there is a risk of not being able to see the flash.

- This unit is not suitable for outdoor use. Do not allow the unit to become damp or wet.
- Only use units of the *Lisa-Signolux* system where electronic devices are permitted.
- When using in aircraft, it is essential to obtain the prior consent of the flight personnel.
- Strong interference signals from third-party devices can impair or completely prevent the function. Do not place potentially interfering devices such as screens and mobile phones closer than 30 cm to *Lisa-Signolux* devices and, if necessary, their cabling, as important event messages such as fire warnings may be prevented.

For your safety: To ensure the function of the system, a function test must be carried out after installation.


If smoke detectors are integrated into the system, it is a legal requirement to check the detectors and their alerting regularly!

Mounting the transmitter

The Baby Transmitter can be mounted using the supplied wall mount. In this case, screw the wall mount to a wall (or wooden beam, cabinet, etc.).

When attaching to walls, it is necessary to use a power drill (observe the manufacturer's safety instructions!) together with a masonry bit (6 mm).

In this case, use the dowels supplied to fix the mount. After attaching the wall mount, hook the transmitter onto the mount.

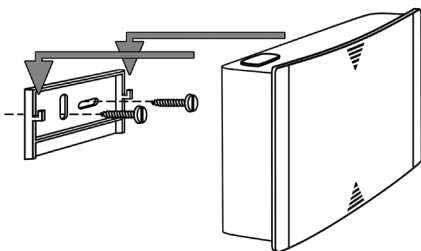Environmental conditions

Units of the *Lisa-Signolux* system should only be operated in a dry environment within the guideline values for temperature and humidity specified in these operating instructions. Humidity and moisture can lead to failure of the units.

Maintenance and cleaning

The unit is maintenance-free. If it becomes dirty, it can be cleaned with a soft, damp cloth. If necessary, use only commercially available alcohol-based disinfectants.

Never use cleaning agents or disinfectants that contain acetone or hydrogen peroxide or that are bleaching, corrosive or abrasive.

No liquid or moisture may penetrate the openings of the unit.

Warranty

The Baby Transmitter is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the serial number in any event.

Disposal of batteries

The included batteries are recyclable. Please dispose the batteries by using a dedicated battery container or return them to the specialist shop. Please dispose only discharged batteries to ensure environmental protection.

Disposal of used electrical and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electrical and electronic units.

By disposing of this product correctly, you are helping to protect the environment and the health of your fellow human beings. Incorrect disposal poses a risk to the environment and health.

Material recycling helps to reduce the consumption of raw material.

You will receive further information on the recycling of products from your local community, your communal disposal company or your local dealer.

Markings

MD This is the labelling of Humantechnik GmbH for a medical product according to EU 2017/745.

UDI This is the marking for the product identification number to identify the medical device.

SN This is the marking for the serial number to identify the medical device. This is indicated on the device and the packaging.

REF This is the identification for the manufacturer's item number.

This is the marking for manufacturer of the medical device.

This symbol indicates the production date of the medical device.

This is the marking of the country of production of the medical device. This is shown on the device and the packaging.

This symbol indicates distributors for the medical device.

This symbol indicates that the user should read these operating instructions attentively.

This sign indicates warnings that are important for the user as well as important information on use and operational safety.

Temperature range for transport and storage: -5°C to 40°C
Temperature range for operation: 10°C to 30°C, i optimum ambient temperature: 25°C

Humidity during transport and storage: < 90% (non-condensing)
Operating humidity: 15% to 90% (non-condensing)

Atmospheric pressure range during transport, storage and operation: 700 hpa to 1060 hpa

This mark indicates that this appliance does not require sterilisation.

The CE mark on the device confirms conformity with the European requirements of the Medical Device Regulation (2017 / 745 / EU Medical Device Directive (MDR)).

- 2011 / 65 / EC + 2015 / 863 / EC RoHS-Directive
- 2012 / 19 / EC WEEE-Directive
- 2014 / 53 / EC RED-Directive

Complete declarations of conformity are available on the Internet at www.humantechnik.com in the service area.

Hereby, Sarabec Ltd declares that this device complies with all relevant UK Statutory Instruments. The full declarations of conformity for this device are available from:

Sarabec Ltd., 15 High Force Road, MIDDLESBROUGH TS2 1RH. UK.

Sarabec's EU authorised representative: HUMANTECHNIK GmbH, Im Wörth 25, D-79576 Weil am Rhein, Germany

All serious incidents must be reported to the manufacturer and the according local authority)!

Technical specifications may be subject to change without prior notice.

Specifications

 Working range:	10 to 30°C, optimum ambient temperature: 25°C
 Operation:	< 90%, non-condensing (transport, storage and operation)
Protection class:	IP 20
Blackout power supply:	3x size AAA (LR03) batteries at 1,5 V
Power supply:	DC 7.5V / 1500mA
Electrical safety:	Protection class 2
Transmission frequency:	868,425 MHz
Max. transmission power:	<10 mW
Transmission range:	up to 200 m in open spaces, up to 80 m within buildings
Height:	75 mm
Width:	122 mm
Depth:	40 mm
Weight:	128 g (without batteries)

Expected service life of the device: approx. 10 years

Compatible *Lisa-Signolux* receiver devices

This transmitter is compatible with all receivers of the *Lisa-Signolux* product range:

- Receiver S (A-2616-W)
- Receiver T (A-2617-0)
- Portable vibrating receiver (A-2619-0)
- Receiver alarm clock (A-2634-0 / W)
- Flashing lamp (A-2644-W)
- Flashing lamp with Type F (CEE 7/3) socket (A-2645-0)
- Wristband receiver (A-2646-0)
- Receiver T Connect (A-2676-0)

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