HUMANTECHNIK

lisa
signolux
Receiver S

lisa signolux Receiver S

DE

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Seite 2

EN

User manual

Page 16

FR

Mode d'emploi

Page 28

NL

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Pagina 40

FI

Käyttöohje

Sivu 52

DA

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Side 64

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The *Lisa-Signolux* »Receiver S« helps people with hearing problems to cope with everyday life. It enables the perception of acoustic events in the home through visible signals. The »Receiver S« is part of a medical system and is compatible with all devices of the *Lisa-Signolux* alerting system.

Scope of Delivery

Kindly check if all following parts are included:

- | | | |
|-----------------|------------------------------------|---------------------|
| - Receiver S | - 4 rechargeable batteries size AA | - Power supply unit |
| - Wall mount | - 2 screws and 2 dowels | - User manual |
| - Warranty card | | |

If the packaging or the packaging seal (round transparent sticker) is damaged, check the condition of the parts. If parts are missing or damaged, please contact your specialist dealer or the manufacturer directly.


Immediate safety instructions

- Before commissioning and using the device, read this manual carefully and completely. Observe the other warnings and safety instructions on page 22.
- Keep this user manual in a safe place so that it is accessible to other users at all times. Each main user must be able to access it.
- Keep *Lisa-Signolux* devices out of the reach of children! This packaging contains small parts which may present a choking hazard!
- Flashes of light can trigger epileptic seizures in people with photosensitive epilepsy (PSE).


Inserting the rechargeable batteries

The battery compartment is located at the rear. Press on the area with the triangular marking and slide the cover off towards the base of the device.

Now insert the 4 rechargeable batteries supplied. Make sure they are correctly positioned according to the markings. Close the battery compartment again by reinserting the cover and sliding it back. Connect the mains adapter. The initial charging time is approx. 6 hours.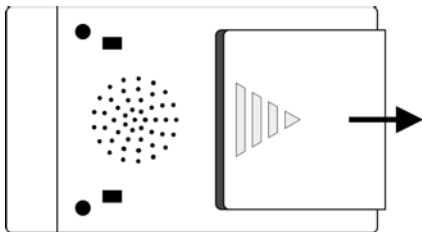

Display and features

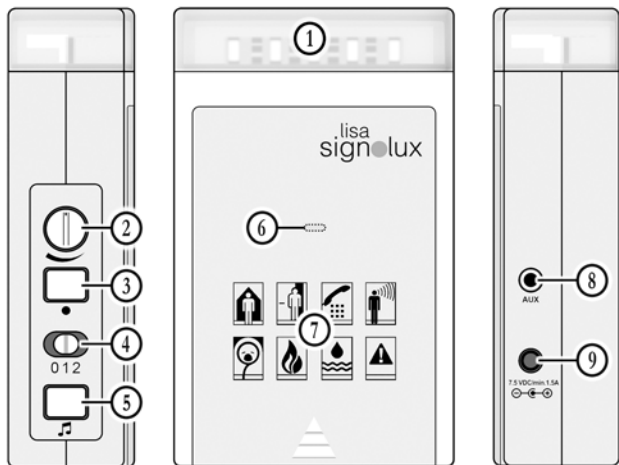- | | | |
|-------------------------|---------------------------|--------------------------|
| 1. Flashing zone | 2. Volume control | 3. Function button |
| 4. Mode selector switch | 5. Melody button | 6. Status LED |
| 7. Event icons | 8. AUX output for modules | 9. 7.5 V power connector |

Function

A *Lisa-Signolux* alerting system consists of at least one transmitter (e.g. doorbell pushbutton) and a receiver.

If e.g. the doorbell pushbutton is pressed, a radio signal is transmitted to the receiver. This, in turn, displays the received signal audibly and visually with sounds and flashing lights.

Setting the alerting mode

Using the mode selector switch (4) you can select the alerting mode.

- switch to position 0: alerting tones, but no flash
- switch to position 1: alerting tones and flash
- switch to position 2: no alerting tones, flash only

Setting the volume

Use the rotating volume control (2) to set the volume of the alerting tone.

Safety warning:

This device can generate sounds of up to 90 dB! Protect your sense of hearing! Please set a low volume to begin with and only then adjust it to the desired volume.

Event icons

The icons symbolize:

Doorbell 1 Colour: Green

Baby Colour: White

Doorbell 2 Colour: Green

Fire alarm Colour: Red

Telephone Colour: Orange

Flood alarm Colour: Blue

Person call Colour: Yellow

Alarm Colour: Red

Automatic pairing

Lisa-Signolux devices can be used together immediately and without prior pairing when delivered.

The displayed icon depends on the type of transmitter. This information can be found in the user manual of the according transmitter.

Optional manual pairing

Note 1:

If a transmitter is to be displayed with an event icon of your own choice, this can be done as described in the chapter „Linking transmitter alerts with event icons“.

However, this causes the receiver to lose the automatic pairing function and all transmitters previously shown on the receiver must be individually paired.

Note 2:

Should there be repeated false alerts when using the preset transmitter indication, this may hint at a neighbouring Lisa-Signolux installation causing interference. In this case, the problem can be solved by individually pairing all transmitter and receiver units within the setup (see „Linking transmitter alerts with event icons“).

Reset to factory defaults

Delete the pairing of each event icon (see „*Deleting transmitters*“).

When all previously assigned transmitters have been removed from the memory of the receiver, the preset transmitter indication is restored and the transmitters are automatically displayed.

Linking transmitter alerts with event icons

Up to 8 transmitters can be assigned to each event icon. This is achieved by pairing a transmitter with an icon of your choice. Please keep in mind that a transmitter can only be paired if it is not already stored in the memory of the receiver. If required, you might first have to delete the transmitter from the memory (see „*Deleting transmitters*“).

- Keep the function button »●« (3) pressed for some time. The event icons start to light up in sequence. Let go of the button as soon as the desired icon is lit.
- Trigger your transmitter (e.g. operate the doorbell pushbutton or test button).
- Wait for about 20 seconds, then test the function by triggering the transmitter again.

Setting the melody

You can freely set every provided tone to an event icon of your choice. The mode selector switch must be either set to 0 or 1.

- First trigger an alert (e.g. operate doorbell pushbutton).
- The icon assigned to the transmitter lights up and the previously chosen tone should be heard.
- Press and hold the melody setting button »♪« (5) for two seconds.
Let go of the button and press it repeatedly until you have found the desired melody (you can choose from 16 different tones).
- Once the icon ceases to flash, the selected tone has been set.

Deleting transmitters

Do the following if you wish to delete a transmitter from the device memory:

- Keep the function button »●« (3) pressed for some time. The event icons start to light up in sequence. Let go of the button as soon as the icon in question is lit.
- Now press the function button »●« and the melody button »♪« (3)+(5) at the same time.
All icons briefly light up and the transmitter is deleted from that icon.

Please note: This procedure will delete **all** transmitters assigned to this event icon.

Status-LED

The receiver informs about the state of its own batteries as well as the batteries of the linked transmitters:

- A sufficient battery charge level of the receiver is indicated by the status LED flashing red every 5 to 6 seconds.
- A low battery level of the receiver is indicated by the status LED flashing red every second. Please recharge the receiver as soon as possible (attach the power supply unit).
- If the battery level of a transmitter is low, the status LED lights up red every 2 seconds after the transmitter has transmitted its signal. Please replace the batteries of the according transmitter at the next possible opportunity.

Behaviour of the status LED when the power supply unit is connected:

Charging the battery: The status indicator lights up in orange and flashes in red every 6 seconds.

Battery is charged and the unit is ready for operation: The status indicator lights up green. Every 6 seconds, the status indicator briefly lights up in red.

Use on rechargeable battery

Mains operation is recommended. The rechargeable batteries are primarily intended to power the unit in the event of a power failure.

The operating time per charge is 2-3 months, depending on the frequency of incoming events.

The receiver should therefore only be temporarily disconnected from the power supply unit.

Recalling the last alert, cancelling the current alert

Press the »●« button to display the last registered alert again.

You can also cancel an active indicated alert by pressing this button.

AUX-socket and power connector

The »AUX«-socket (8) and the power connector for the power supply (10) can be found on the left side of the receiver. All following Humantechnik modules can be connected to the AUX-output:

Vibrating pad (A-3303-0),

Battery-free vibrating pad (A-3304-0/W),

Vibrating contraption (A-3310-0)

These are available as accessories. When an event is displayed, the alert is also indicated over the connected modules.

The power connector is used to supply power via the power supply unit. This is used to charge the batteries, but it can remain connected for permanent operation of the unit.

Positioning the receiver

Important note:


Kindly keep in mind that metal parts and other building materials may interfere with the transmission of the radio signal. As they can be either visible or hidden at the desired mounting location, you should always carry out a functional test at the location in question before starting the mounting process. If required, choose a different placement or mounting location if the reception is not guaranteed.

Mounting the receiver

The »Receiver S« can be set up free-standing or mounted on a wall by using the supplied bracket. In this case, screw the wall bracket to a wall (or wooden beam, cabinet, etc.).

A power drill and masonry drill bit must be used when attaching it to stone walls. Observe the safety instructions of the power drill manufacturer!

In this case, use the dowels supplied to fix the bracket. After attaching the wall bracket, hook the receiver into it.

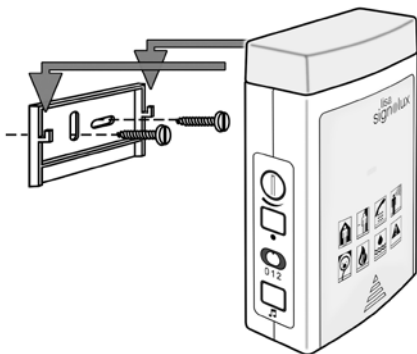Warnings and safety instructions

This chapter lists further important information for safe handling and other conditions for operation.

Safety instructions for rechargeable batteries and power supply

- Charge the batteries fully at room temperature before the first use.
- Only use the specified battery type NiMH AA (HR6)! There is a risk of explosion if the wrong type of battery is used.
- Batteries must not be exposed to high or low extreme temperatures and low air pressure during use, storage or transport at high altitude.
- Disposing of batteries in fires, ovens or by mechanical crushing and cutting may cause fires or explosions. Exposing batteries to extremely high temperatures may cause an explosion or leakage of flammable liquid or gas.
- Batteries exposed to extremely low air pressure may explode or emit flammable liquid or gas.
- Use only the original power supply with the receiver.
- Remove the batteries from the unit if it will be out of use for a longer period of time (> 3 months).
- If the batteries are not sufficiently charged, event messages from the system may be missed!
- Lay the cable of the power supply unit in such a way as to avoid tripping hazards and possible damage to the cable.
- Rechargeable batteries and regular batteries are toxic and must not be swallowed! Keep them out of the reach of children and pets. In case of emergency, seek appropriate medical attention immediately.

Information on product safety

- Please observe these instructions to avoid damage to the unit and possible invalidation of the warranty.
- The device is designed for use in living areas. The unit must not be exposed to direct sunlight for long periods of time and must also be protected from excessive heat, moisture or strong mechanical shocks.
- Only place the unit on flat surfaces. Do not drop the unit. Dropping the unit on a hard surface can damage or destroy it.
- Do not disassemble the device or the power supply unit; there is a risk of electric shock. Opening and modifying the unit will immediately void the warranty. Only use accessories from Humanteknik.
- Do not place the device in extremely bright locations, such as sunlit windowsills, as there is a risk of not being able to see the flash.

- This unit is not suitable for outdoor use. Do not allow the unit to become damp or wet.
- Only use units of the *Lisa-Signolux* system where electronic devices are permitted.
- When using in aircraft, it is essential to obtain the prior consent of the flight personnel.
- Strong interference signals from third-party devices can impair or completely prevent the function. Do not place potentially interfering devices such as screens and mobile phones closer than 30 cm to *Lisa-Signolux* devices and, if necessary, their cabling, as important event messages such as fire warnings may be prevented.

Environmental conditions

Units of the *Lisa-Signolux* system should only be operated in a dry environment within the guideline values for temperature and humidity specified in these operating instructions. Humidity and moisture can lead to failure of the units.

Maintenance and cleaning

The unit is maintenance-free. If it becomes dirty, it can be cleaned with a soft, damp cloth. If necessary, use only commercially available alcohol-based disinfectants. Never use cleaning agents or disinfectants that contain acetone or hydrogen peroxide or that are bleaching, corrosive or abrasive. No liquid or moisture may penetrate the openings of the unit.

Compatible *Lisa-Signolux*-transmitters

The device is compatible with all transmitters of the *Lisa-Signolux* product range:

Direct universal transmitter (A-2655-0/W)	Universal acoustic transmitter (A-2656-0/W)
Doorbell-pushbutton (A-2657-0/W)	Doorbell/Personcall pushbutton (A-2659-0/W)
Smoke detector Guardion (A-2638-0)	Smoke detector Guardion Interlink (A-2642-0)
Combined detector CO/Temp (A-2672-0)	Combined detector CO/Temp Interlink (A-2674-0)
Soundmonitor (A-2680-0/W)	Alarmmonitor „Alarmo“ (A-2681-0/W)

For your safety: To ensure the function of the system, a function test must be carried out after installation.

If smoke detectors are integrated into the system, it is a legal requirement to check the detectors and their alerting regularly!

Service, warranty and warranty conditions

The unit has a high level of operational safety. If faults should occur despite proper operation, please contact your specialist dealer or contact Humantechnik GmbH directly.

The warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Likewise, damage due to natural forces, such as fire and flooding, is not covered by the warranty.

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the serial number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

■ The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products.

Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material.

You will receive further information on the recycling of this product from your local community, your local disposal company or your local dealer.

Battery disposal

Please dispose of used rechargeable batteries and batteries properly, according to your local regulations.

Specifications

 Working range:	10 to 30°C, optimum ambient temperature: 25°C
 Operation:	< 90%, non-condensing (transport, storage and operation)
Protection class:	IP 20
Radio frequency:	868.425 MHz
Transmission range:	up to 200 m in open spaces, up to 80 m within buildings
Loudness:	up to 90 dB at 1 m distance
Rechargeable batteries:	4 x AA NiMH 1.2 V HR6
Power supply:	Primary 100 - 240 V AC / 0.4 A maximum Secondary 7.5 V DC / 1.5 A
Electrical safety:	Protection class 2
Dimensions (H x W x D):	145 x 90 x 40 mm
Weight:	322 g (including batteries)
Operating time per charge:	maximum 1 week
Expected lifetime of the unit:	approx. 10 years

Humantechnik's UK authorised representative:
Sarabec Ltd.
15 High Force Road
MIDDLESBROUGH TS2 1RH
United Kingdom

Sarabec Ltd., hereby declares that this device complies with all UK statutory instruments.
UK declaration of conformity available from: Sarabec Ltd.

Troubleshooting and solutions

Symptom	Cause	Solution
When trying to pair a transmitter, a different icon lights up.	The transmitter you are trying to pair has been already paired with the receiver.	Delete the transmitter from the unwanted icon before linking it with a new icon.
The red status LED lights up with an alert.	This indicates a low battery in the according transmitter.	Insert a new battery in the transmitter.
he receiver does not work.	The batteries may be inserted incorrectly.	Ensure the correct orientation of the rechargeable batteries!
	The batteries are depleted.	Connect the receiver to the power supply.
	The receiver is out of transmission range.	Try using the transmitter or receiver at a different location.
There is no alert tone.	Mode selector switch (4) is on position 2	Check the position of the switch (4) on the rear of the receiver.
		Ensure that the volume is not set to the lowest setting. Rotate the control clockwise to turn up the volume
The flash light does not work.	Mode selector switch (4) is on position 0	Check the position of the switch (4) on the rear of the receiver. .
The transmission range is reduced.	Metal structures and other building materials can reduce the transmission range.	If possible do not mount the transmitter or receiver near or on metal parts.
	Interference from other devices	Avoid setting up <i>Lisa-Signolux</i> devices close to mobile phones and display screens.
	Walls and ceilings reduce the range.	Reduce the distance between transmitters and receivers or try a different position for the devices.
	Low battery charge reduces the range.	Charge the batteries by connecting the power supply unit.

Markings

This is the labelling of Humantechnik GmbH for a medical product according to EU 2017/745.

This is the marking for the product identification number to identify the medical device.

This is the marking for the serial number to identify the medical device. This is indicated on the device and the packaging.

This is the identification for the manufacturer's item number.

This is the marking for the manufacturer of the medical device.

This symbol indicates the production date of the medical device.

This is the marking of the country of production of the medical device. This is shown on the device and the packaging.

This symbol indicates distributors for the medical device.

This symbol indicates that the user should read these operating instructions attentively.

This sign indicates warnings that are important for the user as well as important information on use and operational safety.

Temperature range for transport and storage: -5°C to 40°C
Temperature range for operation: 10°C to 30°C, optimum ambient temperature: 25°C

Humidity during transport and storage: < 90% (non-condensing)
Operating humidity: 15% to 90% (non-condensing)

Atmospheric pressure range during transport, storage and operation: 700 hpa to 1060 hpa

This mark indicates that this appliance does not require sterilisation.

The CE mark on the device confirms conformity with the European requirements of the Medical Device Regulation (2017 / 745 / EU Medical Device Directive (MDR)).

- 2011 / 65 / EC + 2015 / 863 / EC RoHS-Directive
- 2012 / 19 / EC WEEE-Directive
- 2014 / 53 / EC RED-Directive

Complete declarations of conformity are available on the Internet at www.humantechnik.com in the service area.

All serious incidents must be reported to the manufacturer and the according local authority!

Technical specifications may be subject to change without prior notice.

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