HUMANTECHNIK

lisa
signolux

Vibrating Pager

HUMANTECHNIK

lisa
signolux

DE

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Seite 4

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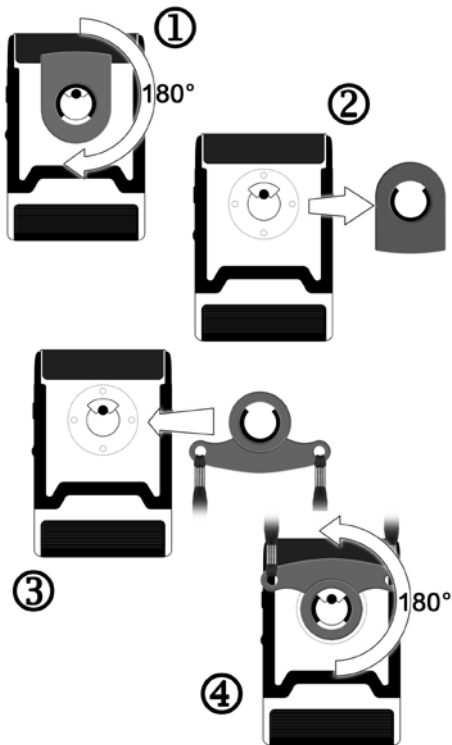
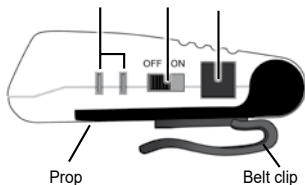
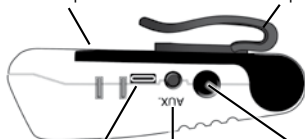
Low
battery
indicator

Battery
compart-
ment

The *Lisa-Signolux* »Vibrating Pager« helps people with hearing problems to compensate for their disability. It enables the perception of acoustic events in the home through vibration and visible information.

The »Vibrating Pager« is part of a medical system and is compatible with all devices of the *Lisa-Signolux* alerting system.

Package contents

Please check if the following parts are included:

- Vibrating pager
- Neck cord
- 2 x 1.5 V AAA batteries
- User manual
- Warranty card

If the packaging or the packaging seal (round transparent sticker) is damaged, check the condition of the parts. If parts are missing or damaged, please contact your specialist dealer or the manufacturer directly.

Function

The vibrating pager is a receiver for the alerts of the various transmitters of the *Lisa-Signolux* alerting system and indicates these as vibrations. 6 indicator LEDs with symbols help to identify the indicated alert.

Immediate safety instructions

- Before commissioning and using the device, read this manual carefully and completely. Observe the other warnings and safety instructions on page 37 and 38.
- Keep this user manual in a safe place so that it is accessible to other users at all times. Each main user must be able to access it.
- Keep *Lisa-Signolux* devices out of the reach of children! This packaging contains small parts which may present a choking hazard!
- The flashes of light emitted by some receivers of the system can trigger epileptic seizures in people with photosensitive epilepsy (PSE).

Comissioning

Inserting the batteries

The battery compartment is on the underside of the pager. Open it by sliding the cover of the battery compartment away from the receiver using both your thumbs.

Insert two 1.5 V AAA batteries and close the receiver again.

Switching the unit on

Set the on/off switch to »ON«. The pager is now ready to operate. After the unit has been switched on, a short self-test lets the battery status LED briefly light up.

Controls and displays

Event indicator LEDs: The 6 indicator lamps help you to differentiate between the individual transmitter signals. They light up for a total of 40 seconds.

Battery status indicator: The receiver informs about the state of the batteries of the receiver as well as the batteries of the transmitters:

- A sufficient battery level of the receiver is indicated by the indicator LED flashing green every 5 to 6 seconds.

- A low battery level of the receiver is indicated by the indicator LED flashing red in short sequence (once per second).

Please replace the receiver batteries as soon as possible.

- A low battery level of a transmitter is indicated by the indicator LED shining together with the symbol of the according transmitter when it's alert is triggered.

Please replace the batteries of the according transmitter as soon as possible.

Function button: An active alert can stopped by pressing this button. It also serves to programm the device.

Belt clip: You can use the belt clip to attach the pager to your clothing. The belt clip can be exchanged with the neck cord (see image on page 46).

On/off switch: Use this switch to turn the pager on and off.

Delete button: This button serves to remove transmitters assigned to an indicator symbol (see „*Deleting transmitters from receiver memory*“).

USB-C charging socket

There is a USB-C charging socket on the side of the receiver. If the receiver is equipped with rechargeable batteries (accessory: 2 x A-4966-0 rechargeable batteries, also included in the A-2910-0 charging station set), the receiver can also be charged using a standard USB charger with a USB-C cable instead of the charging station. During the charging process, the battery status indicator lights up red.

After the charging process has been completed, it lights up green.

The maintenance function of the charge control automatically keeps the batteries at full capacity. This means that the charging process can be active again at irregular intervals, with the battery status indicator lighting up red accordingly.

When operating with normal batteries this charging socket remains without function.

AUX-connector

The following approved accessories (not included in the scope of delivery) may be operated on the AUX-connector:

- Vibrating pad (A-3303-0)
- Vibrating contraption (A-3310-0)

Automatic pairing

Lisa-Signolux devices can be used together immediately and without prior pairing when delivered.

Which event icon is displayed depends on the type of transmitter. This information can be found in the user manual of the according transmitter.

Optional manual pairing

Note 1:

If a transmitter is to be displayed with an event icon of your own choice, this can be done as described in the chapter *„Pairing transmitters and adapting them to the event icons“*.

However, this causes the pager to lose the automatic pairing function and all transmitters previously shown on the alarm clock must be individually paired.

Note 2:

Should there be repeated false alerts when using the preset transmitter indication, this may hint at a neighbouring Signolux installation causing interference. In this case, the problem can be solved by individually pairing all transmitter and receiver units within the setup (see *„Pairing*

transmitters and adapting them to the event icons“).

Reset to factory defaults

Delete the pairing of each event icon (see chapter *„Deleting transmitters from the receiver memory“*).

When all previously assigned transmitters have been removed from the memory, the preset transmitter indication is restored and the transmitters are automatically displayed again.

Pairing transmitters and adapting them to the event icons

Transmitters (e.g. the doorbell pushbutton) can be assigned to an icon of your choice. Up to 8 transmitters can be assigned to each icon.

Please keep in mind that a transmitter can only be assigned if it is not yet known in the memory of the pager.

If required, you might first have to delete the transmitter from its memory (see *„Deleting transmitters from the receiver memory“*).

To assign a transmitter to an icon, do the following:

- a) Keep the function button pressed for some time. The icons begin to light up in sequence. Let go of the function button as soon as the desired icon is lit. (e. g. doorbell symbol)
- b) Trigger the signal of the transmitter you want to assign to the lit up icon. (e.g. operate the doorbell button).
- c) Wait for about 20 seconds and test the function by triggering the transmitter again.

Deleting transmitters from the receiver memory

If you wish to remove the transmitters assigned to an icon from the receiver memory:

- a) Keep the function button pressed for some time. The icons begin to light up in sequence. Let go of the function button as soon as the desired icon is lit.
- b) Now press the function button and the delete button at the same time. All icons will briefly light up.

Please note: This will delete all transmitters from the according icon.

Icon description

doorbell (green)

telephone (orange)

person call (yellow)

babycry (white)

fire alarm (red)

alarm (red)

Recalling the last alert

The receiver stores the last received alert for 3 minutes. During this time, you can recall the latest alert by pressing the function button.

The alert is displayed by the according icon, the vibration is also triggered.

Event identification

The various alert sources can be identified by the different vibration patterns which are unique to each icon:

Doorbell: Doorbell icon (green LED) is lit

Telephone: Telephone icon (orange LED) is lit

Person call: Person icon (yellow LED) is lit

Babycry: Baby icon (white LED) is lit

Fire alarm: Fire icon (red LED) is lit

Alarm: Alarm icon (red LED) is lit

Warnings and safety instructions

This chapter lists further important information for safe handling and other conditions for operation.

Safety instructions for batteries

- Only use the specified alkaline battery type AAA LR03 or the rechargeable battery type NiMH AAA (HR03)! There is a risk of explosion if the wrong type of battery is used.
- Batteries or rechargeable batteries must not be exposed to high or low extreme temperatures and low air pressure during use, storage or transport at high altitudes.
- Disposing of batteries or rechargeable batteries in fires, ovens or by mechanical crushing and cutting may cause fires or explosions. Exposing batteries to extremely high temperatures may cause an explosion or leakage of flammable liquid or gas.
- Batteries or rechargeable batteries exposed to extremely low air pressure may explode or emit flammable liquid or gas.
- Remove the batteries or rechargeable batteries from the unit if it will be out of use for a longer period of time (> 3 months).
- If the batteries or rechargeable batteries do not have a sufficient level of charge, event alerts from the system may be missed!

- Rechargeable batteries and regular batteries are toxic and must not be swallowed! Keep them out of the reach of children and pets. In case of emergency, seek appropriate medical attention immediately.

Information on product safety

Please observe these instructions to avoid damage to the unit and possible invalidation of the warranty.

- The device is designed for use in living areas. The unit must not be exposed to direct sunlight for long periods of time and must also be protected from excessive heat, moisture or strong mechanical shocks.
- Only place the unit on flat surfaces. Do not drop the unit. Dropping the unit on a hard surface can damage or destroy it.
- Do not disassemble the device. Opening and modifying the unit will immediately void the warranty. Only use accessories from Humantechnik.
- This unit is not suitable for outdoor use. Do not allow the unit to become damp or wet.
- Only use units of the *Lisa-Signolux* system where electronic devices are permitted.
When using in aircraft, it is essential to obtain the prior consent of the flight personnel.

- Strong interference signals from third-party devices can impair or completely prevent the function. Do not place potentially interfering devices such as screens and mobile phones closer than 30 cm to Lisa-Signolux devices and, if necessary, their cabling, as important event messages such as fire warnings may be prevented.

Environmental conditions

Units of the *Lisa-Signolux* system should only be operated in a dry environment within the guideline values for temperature and humidity specified in these operating instructions. Humidity and moisture can lead to failure of the units.

Maintenance and cleaning

The unit is maintenance-free. If it becomes dirty, it can be cleaned with a soft, damp cloth. If necessary, use only commercially available alcohol-based disinfectants. Never use cleaning agents or disinfectants that contain acetone or hydrogen peroxide or that are bleaching, corrosive or abrasive. No liquid or moisture may penetrate the openings of the unit.

Compatible *Lisa-Signolux* transmitters

The device is compatible with all transmitters of the Lisa-Signolux product range:

Direct universal transmitter (A-2655-0/W)

Universal acoustic transmitter (A-2656-0/W)

Doorbell-pushbutton (A-2657-0/W)

Doorbell/Personcall pushbutton (A-2659-0/W)

Smoke detector Guardion (A-2638-0)

Smoke detector Guardion Interlink (A-2642-0)

Combined detector CO/Temp (A-2672-0)

Combined detector CO/Temp Interlink (A-2674-0)

Soundmonitor (A-2680-0/W)

Alarmmonitor „Alarmo“ (A-2681-0/W)

For your safety: To ensure the function of the system, a function test must be carried out after installation.

If smoke detectors are integrated into the system, it is a legal requirement to check the detectors and their alerting regularly!

Service, warranty and warranty conditions

The unit has a high level of operational safety. If faults should occur despite proper operation, please contact your specialist dealer or contact Humantechnik GmbH directly.

The warranty covers the repair of the product and returning it to you free of charge. It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Likewise, damage due to natural forces, such as fire and flooding, is not covered by the warranty.

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Always specify the serial number in any event.

Disposal of used electric and electronic devices (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of these products.

Environment and health are endangered by a faulty disposal.

Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Batteries

The included batteries are recyclable. Please dispose the batteries by using a dedicated battery container or return them to the specialist shop. Please dispose only discharged batteries to ensure environmental protection.

Specifications

-  Working range: 10 to 30°C, optimum ambient temperature: 25°C
-  Operation: < 90%, non-condensing (transport, storage and operation)
- Power supply: 2 pcs 1.5 V AAA battery LR03
or: 2 pcs 1.2 V rech. battery HR03
- Protection class: **IP 20**
- Electrical safety: Protection class 2
- Battery endurance (at 3 - 5 alerts per day):
approx. 14 days with 1.5 V alkaline battery
approx. 7 days with 1.2 V rechargeable battery (per recharge)
- Frequency: 868,425 MHz
- Transmission area: up to 200 m in open spaces, up to 80 m within buildings
- Vibration: Eccenter motor, various vibration sequences for identifying alerts
- Height/Width/Depth: 88 mm / 56 mm / 28 mm
- Weight: approx. 98 g
- Expected lifetime of the unit: approx. 10 years

Specifications subject to change without prior notice.

Markings

-  This is the labelling of Humantechnik GmbH for a medical product according to EU 2017/745.
-  This is the marking for the product identification number to identify the medical device.
-  This is the marking for the serial number to identify the medical device. This is indicated on the device and the packaging.
-  This is the identification for the manufacturer's item number.
-  This is the marking for the manufacturer of the medical device.
-  This symbol indicates the production date of the medical device.
-  This is the marking of the country of production of the medical device. This is shown on the device and the packaging.
-  This symbol indicates distributors for the medical device.
-  This symbol indicates that the user should read these operating instructions attentively.
-  This sign indicates warnings that are important for the user as well as important information on use and operational safety.
-  Temperature range for transport and storage: -5°C to 40°C
Temperature range for operation: 10°C to 30°C, i optimum ambient temperature: 25°C
-  Humidity during transport and storage: < 90% (non-condensing)
Operating humidity: 15% to 90% (non-condensing)
-  Atmospheric pressure range during transport, storage and operation: 700 hpa to 1060 hpa
-  This mark indicates that this appliance does not require sterilisation.

 The CE mark on the device confirms conformity with the European requirements of the Medical Device Regulation (2017 / 745 / EU Medical Device Directive (MDR)).

- 2011 / 65 / EC + 2015 / 863 / EC RoHS-Directive
- 2012 / 19 / EC WEEE-Directive
- 2014 / 53 / EC RED-Directive

Complete declarations of conformity are available on the Internet at **www.humantechnik.com** in the service area.

All serious incidents must be reported to the manufacturer and the according local authority!

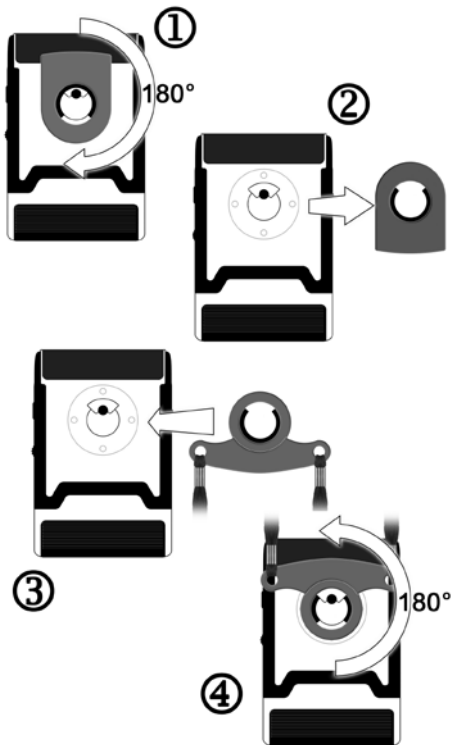
Humantechnik's UK authorised representative:

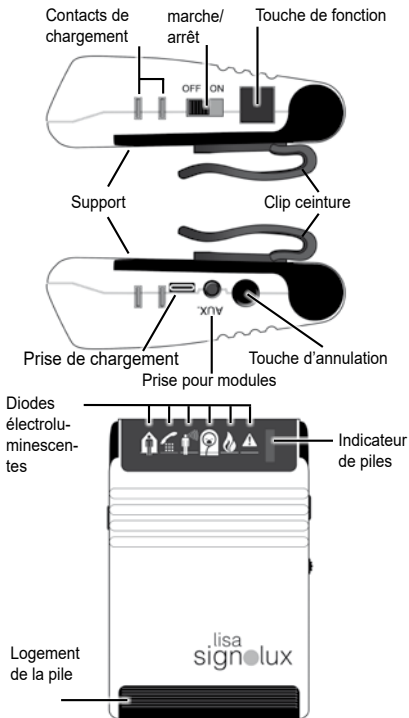
Sarabec Ltd.
15 High Force Road
MIDDLESBROUGH TS2 1RH
United Kingdom

Sarabec Ltd., hereby declares that this device complies with all UK statutory instruments.

UK declaration of conformity available from:
Sarabec Ltd.

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Humantechnik Finland Oy