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Version 2.1 – 23.06.2015

1 Safety Information

For safety reasons do not use this product near machinery as neckloop can become tangled in moving parts.

Intended use

The NL200 induction loop has been developed to establish a connection with Bluetooth®-compatible mobile phones. When connected to a Bluetooth® phone, it can be used as a handsfree communication device for people who wear a hearing aid. Any other use is considered unintended use.

Unauthorized modification or reconstruction is not permitted. Under no circumstances open the device or complete any repair work on it yourself. Please read this operating instruction manual thoroughly.

Compatibility with hearing aids

The product is compatible with most popular hearing aids on the market. However, due to the wide range of hearing aids, there is no guarantee that the product will operate trouble-free with all hearing aids.
(HAC Hearing Aid Compatible to T3 Rating)

Attention: If you have a pacemaker implanted and want to use the NL200 induction loop, please consult your doctor first. Switch the induction loop off immediately if you notice or suspect any interference.

Power adapter plug

Attention: Only use the power adapter plug supplied because other power adapter plugs could lead to equipment damage. Ensure access to the power adapter plug is not obstructed by furniture or such.

GB

Disposal

In order to dispose of your device, take it to a collection point provided by your local public waste authorities (e.g. recycling centre). According to laws on the disposal of electronic and electrical devices, owners are obliged to dispose of old **electronic and electrical devices** in a separate waste container. The adjacent symbol indicates that the device must not be disposed of in normal domestic waste!

Batteries represent a hazard to both health and the environment!

Never open, damage or swallow batteries, nor allow them to get into the environment. They may contain toxic and ecologically hazardous heavy metals.

You are legally obliged to dispose of batteries at the point of sale or at appropriate collection points provided by your local public waste authorities. Disposal is free of charge. The adjacent symbols indicate that batteries must not be disposed of in normal domestic waste and but at authorised collection points.

Packaging materials must be disposed of according to local regulations.

2 Starting Up

Checking the package contents

The package contains:

Content	NL200
Bluetooth® induction loop	1
Power adapter plug	1
Operating manual	1

Charging the battery

1. Connect the power adapter plug to the connection socket (7).
2. When the NL200 is charging, the Status LED (3) lights up red.
3. When the NL200 is fully charged, the Status LED (3) does not light up.

The charging time is 5 hours.

To ensure the functionality of the device even if used irregularly, the installed rechargeable battery must be completely charged at least every 3 months!

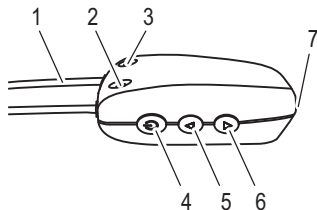
Only use the power adapter plug supplied labelled with output data 5 V_{DC}, 200 mA.

Low battery charge

When the battery charge in the NL200 becomes low, an acoustic alert signal is issued during the call.

Operating elements

- 1 Induction loop
- 2 Microphone
- 3 Status LED
- 4 Power button
- 5 Volume Up key
- 6 Volume Down key
- 7 Power adapter socket

3 Using the NL200

The NL200 implements an induction loop system which operates together with your hearing aid. In order to be able to use the NL200, you must hang it around your neck and establish a connection to your mobile phone.

It is essential to pay attention to the safety information in Chapter 1.

The individual functions provided by the NL200 are described below.

Registering the NL200 on the mobile phone

Prior to using the NL200 for the first time, it must be registered on the relevant mobile phone.

1. Switch on your mobile phone's Bluetooth[®] function.
2. Press and hold the Power button (4) on the NL200 until the Status LED (3) flashes blue and red alternately.

3. Activate the search function for Bluetooth[®] devices on the mobile phone.
4. Wait until the mobile phone has found the NL200.
5. The mobile phone should now display the device with the name "KSY-01".
6. Select this device and enter the code "0000".
7. If registration has been completed successfully and the NL200 is linked to your mobile phone, the Status LED (3) flashes blue.

Switching the NL200 on

The next time you intend to use the NL200, there is no need to complete the procedure for registration on the mobile phone again. After the NL200 has been switched on, it is automatically registered on the relevant mobile phone.

The Bluetooth[®] function must be activated on the mobile phone.

1. Press and hold the Power button (4) on the NL200 until the Status LED (3) lights up blue.
2. A short time later, the Status LED (3) flashes blue and the NL200 is registered on the mobile phone.

Switching the NL200 off

1. Press and hold the Power button (4) on the NL200 until the Status LED (3) no longer lights up.

Increasing the volume

1. During a call, press and hold the Volume Up button (5) until the required volume is set.
2. An acoustic alert signal is issued on reaching the maximum volume.

Reducing the volume

1. During a call, press and hold the Volume Down button (6) until the required volume is set.
2. An acoustic alert signal is issued on reaching the minimum volume.

Canceling the dialling process / Ending a call

1. Press the Power button (4) briefly to cancel the dialling process and/or end the call.

NOTE - due to the wide variety of Bluetooth mobile phones and other communication devices, when your device is used with the NL200 it may be necessary in some cases to end calls from your handset directly. In some devices ending the call through the NL200 would stop the connection of the NL200 with your device, but may not end the call. To avoid call charges and any inconvenience please test your device for compatibility of this function and if required, end calls from your device directly.

Taking a call

1. Press the Power button (4) briefly to take a call.

Voice command

You can control the voice command functions of the mobile phone via the NL200. A condition for this is that your mobile phone supports the functions. For more information, refer to the operating manual provided with your mobile phone.

1. If the NL200 is in Standby mode, press Power button (4) briefly.
2. The mobile phone now receives the voice commands via the microphone in the NL200.

4 Maintenance / Guarantee

Maintenance

- Clean the housing surfaces with a soft, fluff-free cloth.
- Never use cleaning agents or solvents.

Guarantee

AMPLICOMMS equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble-free functioning and a long service life. The terms of guarantee do not apply to the batteries or power packs used in the products. The period of guarantee is 24 months from the date of purchase.

All deficiencies related to material or manufacturing errors within the period of guarantee will be redressed free of charge. Rights to claims under the terms of guarantee are annulled following tampering by the purchaser or third parties. Damage caused as the result of improper handling or operation, normal wear and tear, incorrect positioning or storage, improper connection or installation or

Acts of God and other external influences are excluded from the terms of guarantee. In the case of complaints, we reserve the right to repair defect parts, replace them or replace the entire device. Replaced parts or devices become our property. Rights to compensation in the case of damage are excluded where there is no evidence of intent or gross negligence by the manufacturer.

If your device does show signs of a defect within the period of guarantee, please contact the sales outlet where you purchased the AMPLICOMMS device, producing the purchase receipt as evidence. All claims under the terms of guarantee in accordance with this agreement can only be asserted at the sales outlet. No claims under the terms of guarantee can be asserted after a period of two years from the date of purchase and hand-over of the product.

Declaration of Conformity

This device fulfils the requirements stipulated in the EU directive:

1999/5/EC directive on radio equipment and telecommunications terminal equipment and the mutual recognition of their conformity. Conformity with the above mentioned directive is confirmed by the CE symbol on the device.

To view the complete Declaration of Conformity, please refer to the free download available on our web site

www.amplicomms.com

Important Notes on the Guarantee

The period of guarantee for the AMPLICOMMS product begins on the date of purchase. Rights to claims under the terms of guarantee relate to deficiencies resulting from material or production faults. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, normal wear and tear, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee.

CLAIMS UNDER THE TERMS OF GUARANTEE MAY ONLY BE ASSERTED TO THE RETAILER WHERE THE EQUIPMENT WAS PURCHASED, WHEREBY THE PURCHASE RECEIPT MUST ALSO BE SUBMITTED.

TO PREVENT UNNECESSARY LONG DELAYS, PLEASE DO NOT SEND EQUIPMENT UNDER COMPLAINT TO OUR OFFICE ADDRESS.

IN THE CASE OF QUESTIONS CONCERNING OPERATION OR THE TERMS OF GUARANTEE, PLEASE REFER TO THE OPERATING MANUAL OR CONTACT OUR HOTLINE: 0844 800 6503 (Using an UK landline, at the time of going to print: calls cost 6p for connection charge and then 5p a minute. Mobile costs may vary.)

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AUDIOLINE GmbH