**iLoop<sup>TM</sup>+**

# Neck Loop Boucle Magnétique Induktionsschleife

English P1

Français P11

Deutsch P21

## 1. GENERAL

---


The mobile iLoop™+ inductive loopset is for people with hearing aids.

When you use the iLoop™+, you can speak directly to the caller by the microphone inside the system. So, the device works as a wireless system. Compatible with products equipped with a standard 3.5mm coaxial microphone, headset socket & with most cellular like iPhone, Blackberry, Samsung etc, and DECT.... in some instances you may need to purchase an adaptor in case the product does not have a suitable headset socket.


**NOTE:**

1. Don't forget to switch your hearing aid to *T* mode.
2. *iLOOP™+* can be charged by connecting the USB cable to a PC or standard USB charger.
3. Place the **power ON/OFF** switch on the side of the unit to **OFF** position when not in use to avoid discharging the batteries. The batteries can charge when the switch is in **OFF** position.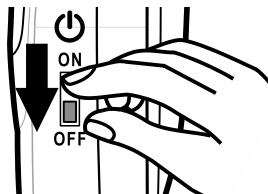

4. Any interference or echo when using your *iLoop™+* could be due to interference from your telephone's headset socket. In this case please use the jack adapter that is included in the box.
5. With the device connected to a mobile phone, pressing the '**Hook On/Off**' button allows you to answer an incoming call.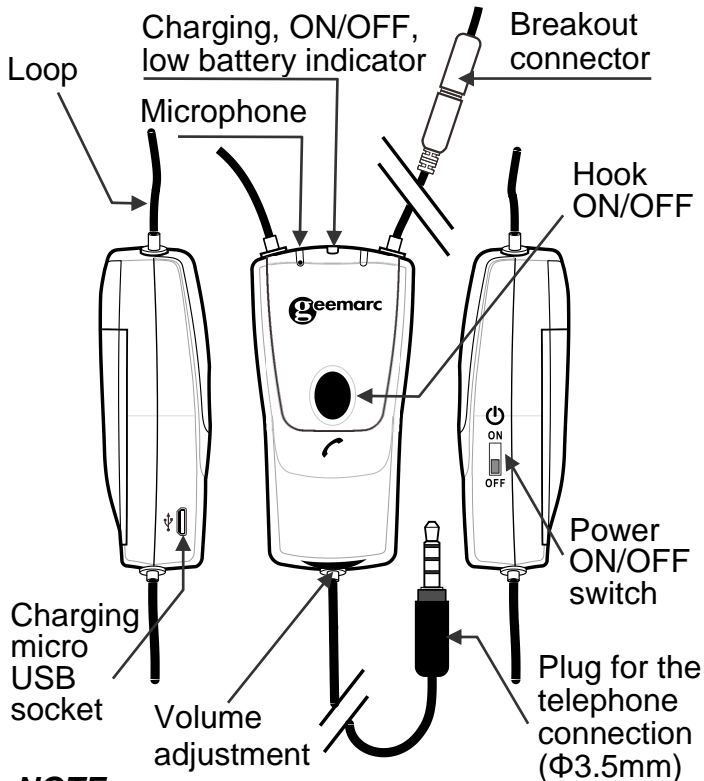

**NOTE:**

- **OFF:** LED is off
- **ON:** LED is Green
- **Low battery:** LED is Red
- **Battery Charging:** LED is Blue

## 2. POWER SUPPLY / CONNECTING

---

The unit needs 2pcs AAA size 1.2V rechargeable NiMH batteries for correct functioning. The batteries must be charged for about 7 hours before use.


- Open the battery compartment door.
- Insert the batteries supplied with the unit, making sure the polarity is according to the diagram inside the battery compartment.
- To charge the batteries connect the supplied USB cable to the Micro USB socket of the unit and then onto the USB of PC or standard power supply. The blue LED will light up.
- Continue charging until the LED switches OFF.
- Disconnect the USB cable after the charging is complete.
- Connect the iLoop™+ in the 3.5mm headset jack of your mobile telephone. If it works correctly and answers the call then this means that your socket on your mobile phone is configured in a way to work in sync with the Geemarc iLoop+. If you find that the iLoop™+ does not work correctly or if there is an echo,

then please attach the jack included in the gift box to plug into the 3.5mm headset socket.

- You can adjust the volume using the volume knob on the iLoop™+. The same instruction regarding the use of the adaptor applies to using the iLoop™+ with tablets, MP3, kindles etc.

### **NOTE:**

- *Always disconnect the plug of the iLoop™+ from the telephone and place the **Power On/Off** switch on the side of the unit to **OFF** position, before installing the batteries.*
- *When the batteries are discharged, the **Red LED** will light up. Charge the batteries immediately to avoid damage to the unit or the batteries.*
- *Check that the microphone/headphone socket is compatible with the 4 contact plug (speaker R & L, common, mic). Otherwise if you encounter some problems you can connect the small jack*

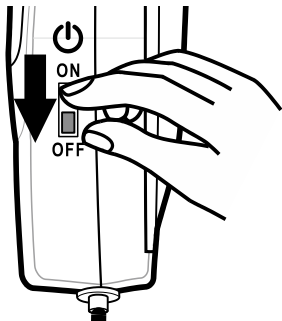
*adapter provided, so as to have compatibility with plug (speaker R & L, mic, common).*


- *Do not dispose off batteries in household rubbish. Pay attention to the battery regulations regarding battery disposal.*
- **When using your iLoop™+ with GSM phones, please note the following:**
  1. *When the audio cable is plugged into older models of simple GSM phones, the Green LED stays OFF. To check if the cable has been plugged in properly, pressing any number button on the GSM phone will switch ON the Green LED.*
  2. *When the audio cable is plugged into the newer Smart phones, the Green LED will switch ON to confirm that it has been connected properly.*

### 3. MAKING / RECEIVING CALLS

---

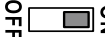- Place the loop around your neck. Switch the **Power ON/OFF** switch  on the side to **ON** position.

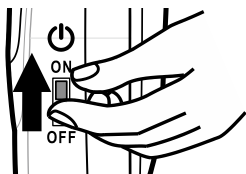- Switch your hearing aid to **T** mode.

- Press the **HOOK ON/OFF** button. You are ready to speak through your iLoop™+.

- The **Green** LED will turn ON.
- During the call you can adjust the volume of your device by using the volume control knob located at the bottom of the iLoop™+.

#### **NOTE:**

- *The Breakout connector will release if the unit gets entangled with the cables, so as to avoid injury to the neck. If this happens, plug back the connector in the jack to resume normal operation.*

## 4. GUARANTEE

---

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our option) are free of charge. Should you experience a problem then contact our helpline or visit our website at [www.geemarc.com](http://www.geemarc.com). The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

**IMPORTANT:** YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

**The guarantee applies to the United Kingdom only.**

**DECLARATION :** Geemarc Telecom SA hereby declares that this product is  in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 2014/53/UE.

*The declaration of conformity may be consulted at [www.geemarc.com](http://www.geemarc.com)*

## 5. RECYCLING DIRECTIVES

---

**Recycling directives:** The WEEE (Waste Electrical and Electronic Equipment) Directive has been put in place so that products at the end of life are recycled in the best way.

When this product is out of order, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.
- Or hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.

For product support and help visit our website  
at [www.geemarc.com](http://www.geemarc.com)

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